<b>HANDLEIDING VOOR GEBRUIK</b>	<b>3</b>
<b>GEBRAUCHSANLEITUNG</b>	<b>10</b>
<b>NOTICE D'UTILISATION</b>	<b>18</b>
<b>MANUAL BIRTH ALARM</b>	<b>26</b>
<b>MANUAL BIRTH ALARM</b>	<b>33</b>

# BIRTH ALARM

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## 1. INTRODUCTION

Thank you for purchasing BIRTH ALARM. BIRTH ALARM is an extremely reliable alarm system for foal births. Before you start to use the system we recommend that you read this manual through carefully, since proper assembly and operation are essential for the system to function correctly.

### United Kingdom

T +44 (0)247 647 2825

E info.uk@gallaghereurope.com

## GUARANTEE

For 2 years after the date of purchase. Does not apply to breakage, water damage and/or other types of damage. Neither the manufacturer nor the importer are liable for any damage whatsoever resulting from the use of BIRTH ALARM.

BIRTH ALARM® is patented and it is a registered trademark.

## 2. ASSEMBLY AND INSTALLATION

BirthAlarm has two parts: the transmitter and the receiver. The transmitter must be mounted on the anti-roll bar. Also the receiver should be installed indoors. All the parts and fittings are included in the packaging.

Assembling the transmitter (on the anti-roll bar)  
Pull the three (or five if required) tie clips through the gaps in the transmitter section. Make sure the ribbed side of the tie clips is on the inside. You can feel this with your hand.

Make sure that the closing system on the tie clip is not on the antenna side or on top of the rolling bar. By this you can prevent the tie clip from being damaged, for example when the mare rolls over.

Place the transmitter upright on the anti-roll bar. The flat base of the transmitter **MUST BE ABSOLUTELY** horizontal.

Attach the two hose clamps only around the anti-roll bar of the girth, in such a way that they are pressed right up against the two lowest tie clips. If this is done the transmitter will be firmly attached and it can no longer shift out of position.

Cut or snip off the loose ends of the tie clips leaving 0.5cm.

Undo the 10 cross-head screws at the front of the transmitter case. Attach the 9V battery which is included, and press it firmly onto the battery contact clip to ensure a good connection. Then place the battery in the transmitter. Then tighten the screws again and make sure the packing on the lid is in place. If assembled properly, the transmitter is now watertight.

The transmitter is now ready for use.

### On and off

There is no on and off button on the transmitter. From the moment the battery is connected the transmitter is and remains in the stand-by mode but in so doing only uses a minimum amount of power.

The power used in the "rest" or "stand-by" position is so small that a full battery can last up to around 3 months. However you are strongly

recommended to replace the battery after each foal birth before you start to use BIRTH ALARM on another mare.

The transmitter only transmits when it is vertical (in other words as soon as the mare lies down completely flat). The green signal light on the transmitter will then come on.

In order to prevent the battery from being unnecessarily drained which might cause leakage, it is advisable to take the battery out of the transmitter if it is not being used for a long period. Otherwise always hang up the transmitter in a horizontal position when it is not being used.

### **Tip – how to use the girth**

It is best to attach the transmitter with the antenna facing the tail end of the horse.

Make sure the girth is well positioned. The girth should be just a bit less tight than a saddle girth. If the girth is too loose, the Birth Alarm can slip partly off the mare's withers when she is lying against the wall. This results in false alarms or no alarms at all.

Get the mare accustomed to the girth beforehand. Use the supplied belly strap (or a soft alternative). The girth might move to just behind the front legs because of the shape of the pregnant horse. If any irritation arises use a girth protector. Take the girth off the mare regularly. Clean the girth and brush and tend the skin. It is particularly important not to forget this in the spring when the mare is moulting.

For miniature Shetland ponies it may be necessary to fit a tail strap to the girth so that it will stay in place.

The anti-roll bar with transmitter can stay on during foaling.

### **Installing the receiver**

Place the receiver in the best possible position (= where good reception can be achieved). The higher up the receiver is located, the better the

reception, preferably in front of a window which looks out in the direction of the transmitter. The maximum range of BIRTH ALARM is 1000 metres. Screw the antenna of the receiver in the opening on the upper right-hand side of the receiver. Stick the adapter into the stop contact and the connector at the other end of the adapter lead into the plug box no. 6 at the rear of the receiver.

### **3. OPERATION AND FUNCTIONING**

The receiver comes with a single push button (1). All receiver functions are operated using this push button.

The receiver can be switched on by gently pressing briefly on the button and switched off again by pressing a little longer (2 seconds) on the button.

BIRTH ALARM has 3 different user programmes and once the receiver is on you can simply press to switch from the user mode. Which mode is active is shown on the lights (a), (b) and (c). The meaning of this is described briefly on the receiver. See below for more user information on these programmes.

#### **Programme (a)**

##### **Mares resting the normal way**

Pressing briefly on the button allows you to choose one of the three modes. Once the programme is chosen, a green light comes on. MAKE SURE THE RIGHT LIGHT COMES ON. For normal resting heavily pregnant mares we choose programme (a).

As soon as delivery starts and the contractions begin, the mare lies down flat. This causes the transmitter on the anti-roll belt to go to the vertical position, and the system starts working. In this programme (a) after around 7 seconds the buzzer alarm will go off. Along with the buzzer alarm the red light (3) flashes. After around 2 minutes the buzz alarm stops. Light 3 stays flashing.

If there is an alarm or there has been one, you will need to reset the receiver as follows (to bring the receiver back to non-alarm mode)

This procedure applies to all mode positions (a), (b) and (c).

Press button (1) to switch off the receiver. Leave it off for 25 seconds before turning it on again by pressing button (1) again. The receiver is now ready for the next alarm.

(In practical terms it means that you, when warned by the buzzer alarm, turn off the receiver, go to the mare and turn on the receiver again if you wish to use it again)

### **Programme (b) Mares which rest horizontally**

The alarm does not usually go off with resting mares. In setting (a) you would get a false alarm in this case. Birth Alarm has developed a special "sleep mode" for this group: you can use this mode if you notice that you are getting false alarms in the normal mode (a) because the mare is resting horizontally.

You switch on the sleep position by pressing once or several times briefly on button 1 until light (b) comes on

The system then works as follows:

Your horse is resting flat on her side. Since you have set the receiver to mode (b), the alarm does not go off. If the mare gets a contraction (within 2.5 minutes after lying down in that position), she will have rest at the end of the contraction. Therefore she will have to leave her resting position (flat on her side). As soon as the transmitter registers this movement the alarm will go off within 1 second.

Are you not certain whether your mare belongs to this exceptional group or not? Test Birth Alarm on your mare: if there are a lot of false alarms, switch over from setting (a) to setting (b).

Where mares are restless in their sleep, and the receiver is on setting (b), there is a higher chance of false alarms.

## **Colic Guard**

You can also use Birth Alarm as a warning system for horses suffering from colic. To choose this mode, press button (1) until pilot light (c) comes on.

The device now works as follows:

If your horse or pony gets a severe attack of colic and during this attack of cramp is lying flat on the floor for longer than 2 seconds, the receiver's alarm goes off.

## **4. FUNCTIONAL CHECK**

### **Check the battery**

The Birth Alarm has an built-in battery check. If the transmitter is held vertically, the green control light should stay on continuously ("transmitting"). If the light constantly flashes fast, this indicates that the battery has not enough power left. If nothing at all happens, the battery is of course completely dead.

Only use alkaline batteries (Duracell) or batteries with an equivalent energy curve. Never use rechargeable batteries. If in doubt, replace the battery immediately. Under normal usage conditions the battery will last for several weeks and this damage is excluded from any warranty.

If you do not use BIRTH ALARM for a long period of time, take the battery out of the transmitter. Leaking batteries can do irreparable damage to the transmitter's printed circuit board.

### **Checking the reception of the sender**

Check regularly (at least 3x per week) whether the receiver is receiving the signals from the transmitter. In this way you can test whether the receiver is in the right place and whether the equipment is intact.

### **Checking procedure**

Set the receiver to mode (c). Take the transmitter in your hand and go to the furthest place which is accessible to the mare. - Hold the transmitter vertically for few seconds, near the ground. If

everything is in order, the buzzer alarm will go off. If this does not happen, then this location is outside the transmitter range.

NB: if the alarm on the receiver goes off, then this can be switched off by switching off the receiver for a minimum of 25 seconds. The receiver can be switched off using button (1).

### Automatic transmitter monitor

BIRTH ALARM comes complete with a so-called automatic transmitter monitor. NB: This function is only available in transmitters with green housing, not in any transmitters with black housing purchased.

The result of this transmitter monitor is that the range and operation of the transmitter is constantly monitored. The presence of the transmitter is shown by the fact the green pilot light (2) on the receiver comes on. NB: after the receiver has been switched on, it can take a few minutes for this green light to come on.

When the transmitter is out of range or no longer works as a result of perhaps a completely dead battery, after a few minutes the receiver will sound the alarm by emitting an alarm sound every 2 seconds.

If the receiver emits an alarm sound every 3 minutes, this is a sign that the battery is starting to get empty and must be replaced immediately.

NB! As mentioned the transmitter monitor is only active when a receiver is used in conjunction with a transmitter in a green housing. 2 or more transmitters must never be used in green housing at the same time, as in such cases the transmitter will not sound the alarm until all transmitters have stopped at the same time.

Every extra transmitter that you use simultaneously should therefore be one with black housing. This will not be automatically checked for function and range. These transmitters do return to normal and will transmit if the horse lies down.

You must check these extra transmitters yourself

periodically in the manner described above.

### Channel code settings

The Birth Alarm has been designed in such a way that neighbouring horse owners can use it without the two systems interfering with each other. In order to do so you have to adapt the channel code on one of the systems. The Birth Alarm has 4 different channel codes. The default setting is channel code no. 1.

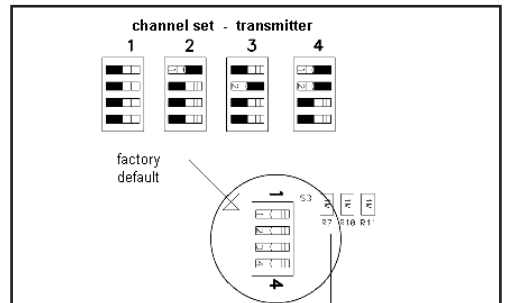
**This modification should be carried out by a authorised Birth Alarm dealer only.**

### IMPORTANT:

in a set (receiver and transmitter) the channel setting must be changed in both the transmitter and the receiver. Please note: if you have extra transmitters, these must also be set to the new channel!

Changing the transmitter channel settings:


1. Unscrew the transmitter housing to open
2. The print plate is now visible, to the top left there is a so-called DIL switch for setting the transmitter code. The small switches 1 and 2 are for channel setting. These are as standard on "OFF" (channel 1)
3. See the drawing below for the settings to be made. NB: never change the positions of switches 3 and 4.

Changing the receiver channel settings:

1. Carefully remove the 4 rubber adhesive feet on the bottom
2. Remove the four screws released

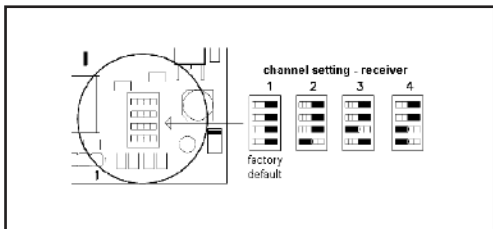- The bottom half of the housing can now be removed.
- The receiver print plate is in the top half.
- See the drawing below for the setting to be made

## 5. OPTIONS

### Multiple transmitters

It is possible to monitor as many mares as possible at the same time using extra transmitters. However, only one (green) transmitter with automatic transmitter monitor may be used. All the others must be in black housing.

### Connecting an extra buzzer or telephone dialler

If you want to install an extra buzzer or telephone dialler, connection 5 on the receiver is provided for that purpose. Use an extension lead (25m) to connect this equipment. Connection 5 switches simultaneously with the alarm signal. Warning: only connect original Birth Alarm peripherals. If in doubt, consult your dealer.

On request, your 230 volt adapter connection can be replaced by a connection to a 12 Volt (car) battery.

## 6. FAQ

### I get a false alarm...

It also might be caused by other transmitting systems in your area such as cordless telephones, baby alarms etc. You can check this by leaving the receiver on for some time without the transmitter on the mare. If you still get an alarm, contact your dealer.

### I don't get an alarm...

Set the receiver to mode (c) for testing purposes. Ensure the transmitter and receiver are working correctly by holding the transmitter in a vertical position for at least 5 seconds. If the alarm does not work, replace the battery and/or check whether the receiver is on or off. Take care to stay within the maximum transmission range. As stated above this may be adversely affected by high-tension power cables, electrical wiring, other (non-approved) transmission equipment, iron foaling boxes, concrete walls etc.

### The transmitter is damaged...

Return the transmitter and receiver to us immediately. We will test the transmitter for you and replace any damaged parts.

### Tips and tricks!

Problems with reception?

Make sure these items are not blocking the antenna reception:

- Leaded windows
- Metal laminated or coated glass
- Closed metal blinds
- Concrete walls with steel reinforcement
- Mobile telephones
- Electrical domestic appliances
- Passing trams, scooters, and trucks
- Changes in the construction around the house
- Steel construction in the shed or barn where the signal is coming from.

### IMPORTANT

All systems bought before 2006 are not compatible with this system.

If you already have BIRTH ALARM components (transmitter, receiver) in stock and wish to use these in combination with this system, first check if these are suitable, as only BIRTH ALARM components working on 433 MHz-band, can be combined with this system. Whether a BA-transmitter or receiver works on 433 MHz, you can see for example by the:

- type- and frequency indication (resp. H III-B

and 434 MHz) on bottom of transmitter and rear of receiver.

- rubber antenna (both transmitter and receiver)
- green LED on the transmitter (on old systems this is red)

## 7. GENERAL

Warning: exposing the Birth Alarm to water is disastrous for the sensitive technology inside Birth Alarm.

You can avoid damage to the transmitter, for example by making sure that the transmitter is out of reach of other horses.

## 8. TIPS

A pregnant mare normally gives birth when there is nobody around. This is a natural behaviour of horses. A foaling stall (for a single horse) must be at least 3.5 square metres and must not contain any obstacles or protrusions which might damage the transmitter and/or the horse.

It is well known that foaling out of doors is always preferable for reasons of hygiene, space etc. Warning: Avoid sloping or uneven ground if you let the mare move around freely outside, since sloping or uneven ground may have a negative effect on the function of BIRTH ALARM.

If the alarm sounds and the birth is about to happen, try to leave the mare alone by watching the birth via CCTV or similar.

Check if the front legs of the foal are in the right position and clean the mouth of the foal after the head becomes visible. However try to leave the mare alone as much as possible until after the birth. The mare will get up by herself. In that way the blood can still flow to the foal via the placenta. After the mare stands up the umbilical cord will brake at the weakest point, a few centimeters from the foals belly.

Don't have to much bedding in the stall. The young foal will have trouble moving about in it.

Have some woodshavings and sand available

close to the stall. After the birth put some woodshavings on the wet spots in the bedding behind the mare. If you are using straw in the stall spread some sand on it. The sand will fall through the straw on the stall floor and prevents the foal from slipping when trying to get up.

Get the mare used to the barn lights you are using during the birth by switching them on on unusual times during the evenings before the birth.

A perfect bedding can be made by using woodshavings in a layer of 10 cm. Prepare this bedding about 2 weeks before the birth.

The advantage is that the mare doesn't get injured by this type of bedding. No slippery floors because it absorbs the amniotic fluids.

The foal gets up easier and learns to walk faster and easier than in a thick layer of straw.

As a result of this the foal is less likely to get injured by repeatedly getting up and falling down.

Things to know:

Birth Alarm is patented in the Netherlands and has been on the market since 1990. Since its introduction more than 80,000 foals have successfully been born using this system.

Birth Alarm is used worldwide and is without a doubt market leader in its area.

Even the very first systems are still being used each year, to the full satisfaction of the users.

Using the Birth Alarm in combination with low cost CCTV systems is a good alternative.

## 9. TECHNICAL DATA

Transmitter – single-channel transmitter

Maximum transmission distance : depending on  
the position of transmitter  
and receiver, max. 1000  
metres

Frequency : 434.650 Mhz

Operational temp. : -10°C tot 55°C

Power supply : 9V – DC (battery)

Power consumption : 30 mA when sending,  
10uA in rest position

Standby

Auto Power Off) : 3 minutes

Dimensions : 13.0 l x 3.5 w x  
9.0 h (cm)

Weight : about 200g

Coding Options : 4 codes

Receiver – single-channel receiver  
with coding unit

Frequency : 434.650 Mhz

Power supply : 10-12V – AC/DC  
(through 230 VAC  
mains adapter)

Power consumption : 20 mA

Sensitivity : 0.5  $\mu$ V

Output extra outlet : zero-potential  
make-and break  
contact (max. load  
1A and max. 24V)  
and external 12V  
DC power supply

Operational temp. : -10°C to 55°C

Dimensions : 18.5l x 12.5wx  
4.5h (cm) excluding  
adapter and antenna

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### of Opinion

N°: 05214195/AA/00

With respect to Chapter 10 of the Telecommunications Act of The Netherlands, Telefication declares that to our opinion the listed product complies with the essential requirements, in accordance with Article 3 of the Directive 1999/5/EC, as indicated under Annex 1 of this statement.

Product description: **Foal monitoring system for horses**  
Trademark: **Birth Alarm**  
Family name: **--**  
Type designation: **H III B**  
Serial N°: **50000 / 50002**  
Software release N°: **V1.0**

Manufacturer: **Skedio Circuits b.v.**  
Address: **Segment 25**  
City: **6921 RC Duiven**  
Country: **The Netherlands**

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Seriennr. \_\_\_\_\_

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# WARRANTY CONDITIONS

The following warranty conditions apply for the Birth Alarm:

1. With the presentation of the certificate of warranty and the purchase note Gallagher provides a 2 year warranty, to be counted from the date of purchase.
2. The warranty comprises the free repair or replacement (of components) of the equipment in the event of material or production faults.
3. The warranty period is not extended or renewed as a result of warranty work activities.
4. Equipment or components of the equipment which are replaced under the warranty automatically become the property of Gallagher.
5. The warranty does not cover the replacement of consumable articles such as batteries, etc.
6. The warranty obligation of Gallagher gives the buyer no sort of right to damage compensation or the dissolving of the purchase agreement.
7. The warranty is no longer applicable if:
  - repairs, changes or extensions to the equipment have been effected by parties other than Gallagher without the permission of the latter;
  - in the opinion of Gallagher the equipment has been neglected or use incompetently;
  - type numbers and/or serial numbers and/or warranty stickers have been damaged, removed or changed;
  - changes have been made to the certificate of warranty or the purchase note;
  - defects have originated due to the incorrect placing of batteries or their leakage;
  - defects have originated due to deviating environmental conditions, as far as such conditions are indicated;
  - defects have originated due to the connections of non-approved equipment;
  - the seal of the equipment, as far as this is present, has been broken;
  - defects have originated due to water damage to the electronics;
  - the housing for the electronics is no longer intact.

You have a two year warranty on Birth Alarm (sender and receiver). Please return warranty card to the above address.

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