

EQUINE LTS HOOF BOOT

Equine LTS Hoof Boot is a treatment tool used exclusively when the horse is tied up and under constant supervision. The processing time is 10-15 minutes. If Equine LTS Hoof Boot is used in the box or if the horse is unattended during treatment with LTS, the warranty lapses. Always stay with the horse during treatment with Equine LTS Hoof Boot to make sure the horse stays calm.

The rechargeable battery

On the top of the battery is a switch, to turn the battery on/off. The switch must be turned ON for the battery to be able to charge. Do not store the battery in a cold place, it will discharge just as a mobile phone. Always charge the battery under constant supervision.

Disconnect the LTS from the battery, while the battery is being charged. The charger for the battery, is only to be used for the battery. Do not use any other charger for the battery.

RECHARGING:

1. Put the recharger in the power outlet. It will turn green.
2. Connect the battery.
3. Switch the battery on. The charger will turn red.
4. When the charger turns green again, the battery is fully charged.
5. Disconnect the battery and make sure to switch off the battery.
6. At temperatures below 5 degrees, the battery will discharge. Make sure to keep the battery warm when not used.

The Hoof Boot:

1. Always ensure that the Equine LTS Hoof Boot is used on a dry and clean area. If necessary, dry with a clean towel first.
2. Place the Equine LTS Hoof Boot on the hoof and connect it to the LTS device via corresponding cable or attach the battery pocket and turn on. Let on for 10-15 minutes while you stay with the horse. Turn off the battery and remove the LTS. Repeat the treatment daily.
3. Max treatment time per area is 10-15 min, may be repeated several times daily in case of laminitis. Total processing time max. 30 minutes, after which the LTS should be cooled. After this, treatment can be resumed.
4. DO NOT leave the horse unattended or in a box unattended during treatment. The warranty is void when used unattended.
5. Do not use any power supplies other than the one provided, as this will void the warranty. That is, if a battery is included, it is this that should be used as a power supply and NOT the charger for the battery, since this is not intended to power the LTS.
6. Do NOT look directly into LED lights.
7. Led lights can be gently cleaned with a cotton swab with rubbing alcohol while it is turned off.
8. DO NOT put the Equine LTS Hoof Boot in the washing machine or otherwise in contact with water, as the electrical system will be destroyed and the warranty will be void.
9. As storage, use only the pouch provided. LTS MAY ONLY BE FOLDED ALONG THE LIGHTS when storing. If folded on the other link, the warranty is void.
10. Do NOT stand, lie, or place anything heavy on top of the LTS. LTS must always be placed on top of the area in question.
11. One must NOT twist LTS. Inside the LTS itself there is an electrical system consisting of wires, very thin copper paper that can break if you twist or fold the LTS the wrong way.
12. The warranty covers manufacturing defects. Cracking of strings during use is not a manufacturing defect and is therefore not covered by the warranty.