

EQUINE LTS R-SERIE MINI PRO, NECK COLLAR

IMPORTANT INFORMATION

Before you plug and use the Equine LTS including the Power Bank, make sure of the following: The Equine LTS including Power Bank is always under constant supervision during use and charging of the Power Bank.

The Equine LTS including Power Bank is only to be used for the normal purposes of a product this type. The

Equine LTS including Power Bank is properly connected to the right type of grounded electrical outlet. The

Equine LTS including Power Bank is not near a heat source such as radiator, oven or similar device. The Equine LTS including Power Bank is out of the weather and located in an area where there is no risk of it becoming partly or fully immersed in water. To switch ON the Power Bank, push the button 1 time – wait for 3 seconds and it will switch on. To turn OFF the Power Bank either push the button twice or hold the button in until it switches off, depending on which model of Power Bank you have received. Connection USB cable to your LTS products included. USB charger not included AT TEMPERATURES BELOW 5 DEGREES, THE POWER BANK WILL DISCHARGE. MAKE SURE TO KEEP THE POWER BANK WARM WHEN NOT USED.

Important and practical information

Equine LTS is a treatment tool that is only used when the horse is tied up and under constant supervision. The treatment time for the Mini pro is 5-10 minutes. If Equine LTS is used in the stable untied or the horse is unattended during treatment with LTS, the warranty is void. Always stay with the horse during treatment with Equine LTS so you can make sure it stays calm.

- Only apply the LTS on a dry clean surface.
- Place the LTS Mini Pro on the injured area and turn it on, for 5-10 minutes. Repeat the treatment daily, for 5-10 minutes. Treatment time for large muscles is 5-10 minutes.
- Do not use for more than 10 minutes at one place. The treatment can be repeated 2-3 times a day.
- The warranty will not cover bending of the light.
- Do not leave the horse out of sight, during treatment.
- Only use the power source, purchased with the LTS.
- Do not look directly into the light.
- Carefully clean the LED light with a Q-tip and rubbing alcohol.
- Do Not wash the LTS.
- Use the carrying bag for storage. Do not fold across the light.
- Do not stand, lay up on or place heavy objects on top of the LTS.
- Do not twist the LTS.